TENS 7000[™]


CMPASSHEALTH


QUICK REFERENCE GUIDE

PAGE 1



The purpose of this guide is to assist you in correctly operating your TENS 7000[™] unit. Do not use this device for undiagnosed pain symptoms until consulting a physician. This guide is not intended to replace or substitute the TENS 7000[™] Instruction Manual. **PLEASE READ THE INSTRUCTION MANUAL COMPLETELY BEFORE USING THIS DEVICE.** Read, understand and practice the contraindications, warnings, cautions and operating instructions found on pages 4 – 5 of your instruction manual. Always follow the operating instructions as listed in the manual.

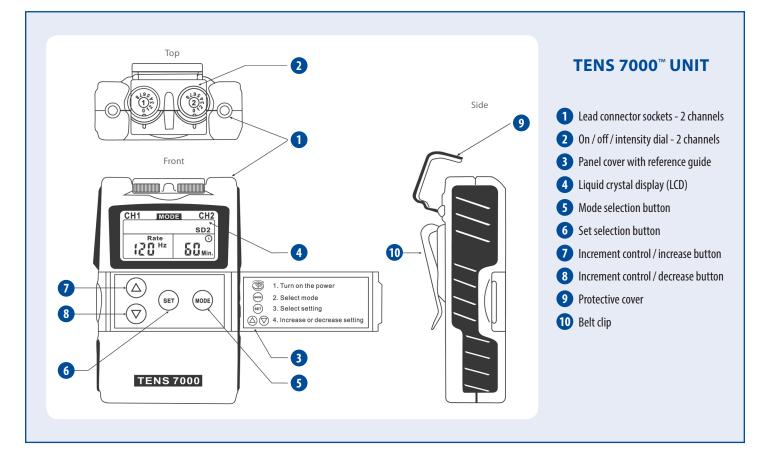

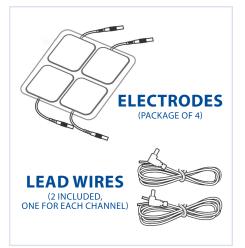


SPECIAL FEATURES

The placement of electrodes can be one of the most important parameters in achieving success with TENS therapy. If the initial results are not positive, speak to your physician about alternative stimulation settings and/or electrode placement. Once desired results are achieved, mark down the electrode sites and settings for future reference. The TENS 7000[™] features two protective covers that prevent unintentional changes to settings.

Turn on device by rotating the dial clockwise and select the desired intensity level. **Turn off device** by turning dial counterclockwise to 0.

Low battery indicator s signals when battery is low. Replace battery when stimulation feels weaker.

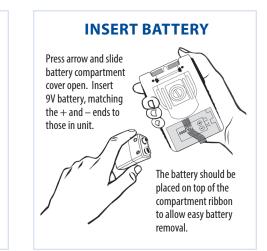


TENS 7000[™]

C()MPASSHEALTH


QUICK REFERENCE GUIDE

PAGE 2

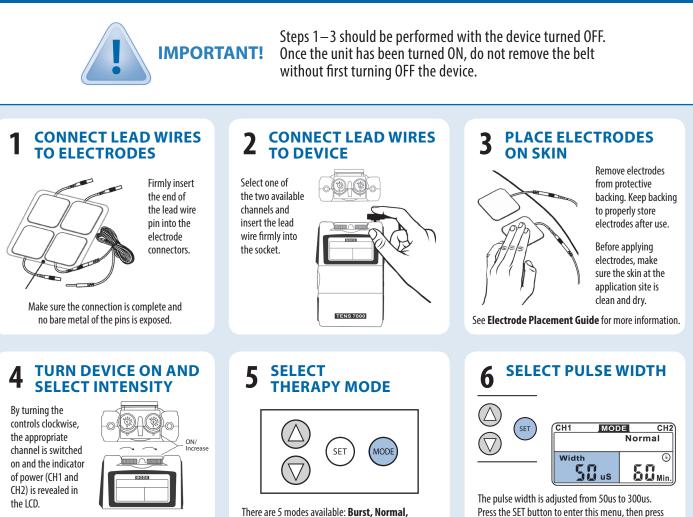

The stimulation output will increase as the dial is turned clockwise or to a higher number. 1 = Weaker 8 = Stronger


7 SELECT PULSE RATE

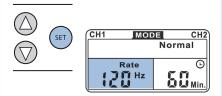

The pulse rate is adjusted from 2Hz to 150Hz. Press the SET button to enter this menu, then press the \blacktriangle and \blacktriangledown buttons to adjust the setting. Unless otherwise instructed, a setting of 70-120Hz is suggested.

There are 5 modes available: **Burst, Normal, Modulation, SD1** and **SD2**. Press the MODE button and continue to press until you have reached your desired setting.

8 SET THE TIMER

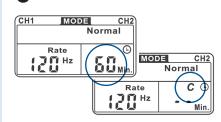

The treatment time is adjustable from 1 to 60 minutes, or Continuous (**C**). Press the \blacktriangle and \bigtriangledown buttons to adjust the setting. Your settings will be stored permanently unless they are adjusted again.

COMPLIANCE METER

This unit can store 60 sets of operation records. The total treatment time can store up to 999 hours.

the ▲ and ▼ buttons to adjust the setting.

is suggested.

Unless otherwise instructed, a setting of 70-120us

To **check and/or delete individual records**, press the MODE button as you turn on the device. The LCD will show the number of records and operation time. Press the \blacktriangle and \checkmark buttons to check each record. To delete a record, press and hold the SET button for 3 seconds.

To **check and/or delete the accumulative record**, press the MODE button to switch to this setting. Press the SET button first, the press MODE and hold for 3 seconds; all of the records will be deleted, followed by the sound of a beep.