


QUICK START GUIDE



Please read thoroughly the User's Manual before operating the unit. Refer to the safety instructions for more information.

1. UNSCREW THE ARRESTING SCREW ON THE BOTTOM OF THE DEVICE

The screw serves to prevent vibrations of the pump during transport. Keep the screw for future use or use it for securing the device to the cart (optional).

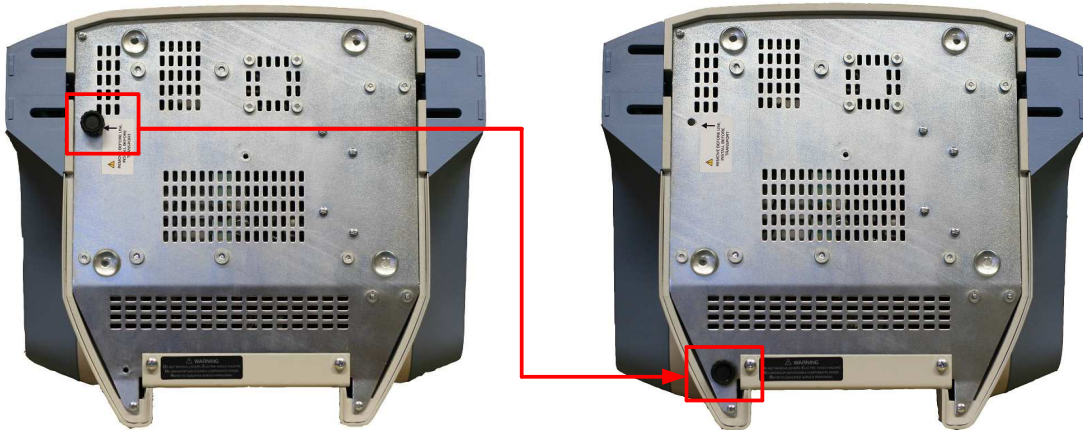

2. CONNECT THE BASIC TUBE

Make sure that hoses are connected properly according to their labels (Hose 1 to Connector Port 1, etc.)

To connect the hose, grip it and plug it into the respective connector and push.

To disconnect the hose, grip the blue ring of the connector with your fingers, push it and then pull the hose out with the other hand.



3. CONNECT THE POWER CORD TO THE UNIT AND INTO POWER OUTLET

4. TURN ON THE MAIN SWITCH ON THE REAR PANEL (TO THE POSITION "I")

5. TURN THE UNIT ON BY PRESSING ON/OFF BUTTON ON THE FRONT PANEL

6. CONNECT THE APPLICATOR

7. PRESS “START” BUTTON TO START THE THERAPY

