


QUICK START GUIDE



Please read thoroughly the User's Manual before operating the unit. Refer to the safety instructions for more information.

1. UNSCREW THE ARRESTING SCREW ON THE BOTTOM OF THE DEVICE

The screw serves to prevent vibrations of the pump during transport. Keep the screw for future use or use it for securing the device to the cart (optional).


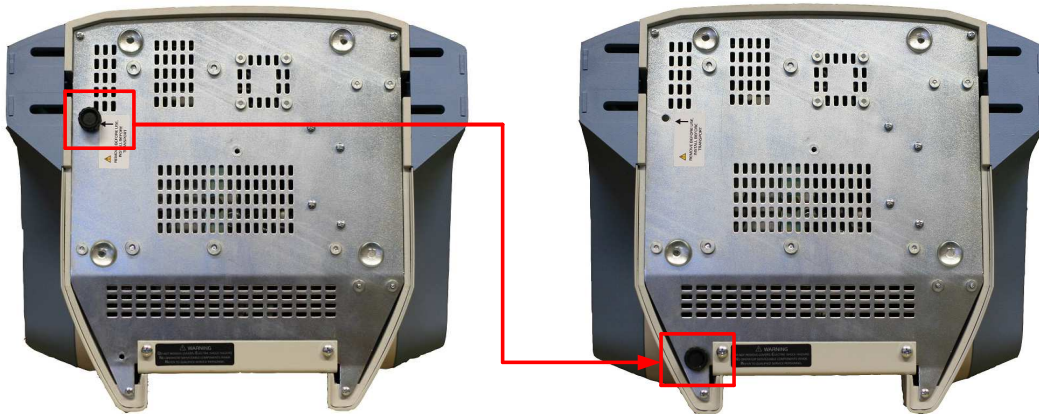


2. CONNECT THE BASIC TUBE

Make sure that hoses are connected properly according to their labels (Hose 1 to Connector Port 1, etc.)

To connect the hose, grip it and plug it into the respective connector and push.

To disconnect the hose, grip the blue ring of the connector with your fingers, push it and then pull the hose out with the other hand.

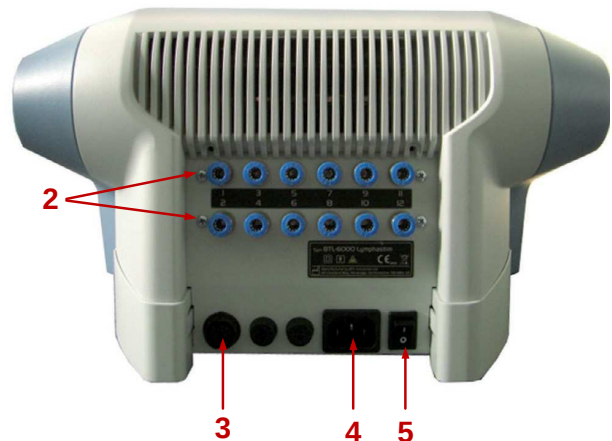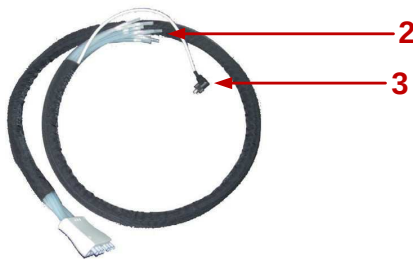

3. CONNECT THE DETECTION CABLE OF THE ACCESSORY TO THE DETECTION PORT

- A) Turn the plastic ring on the cable connector counter clockwise (unlocked position).
- B) Plug the accessory to the main unit (the connector has a dent that only fits one way).
- C) Click-lock it (clockwise).



4. CONNECT THE POWER CORD TO THE UNIT AND INTO A POWER OUTLET

5. TURN ON THE MAIN SWITCH ON THE REAR PANEL (TO THE POSITION “I”)

6. TURN THE UNIT ON BY PRESSING ON/OFF BUTTON ON THE FRONT PANEL

Wait until the unit boots up (a sandglass pictograph appears on the screen).

7. CONNECT THE APPLICATOR

When you connect a new applicator for the first time, confirm the installation of a new accessory by pressing the **yes** button.



Then, select the correct accessory(ies) and press the **install** button. Next time, the device will detect the applicator automatically.



8. PRESS “DIAG” BUTTON AND SELECT A PRESET PROTOCOL

9. PRESS “START” BUTTON TO START THE THERAPY

